

Sleep-eze

Quick Start Sheet

1 Viewing Sleep-eze in detail



When setting the Sleep-eze in your ear, the volume control is at the top, and the engraved serial number faces forward, towards your nose. The right unit has a red dot near the volume wheel and the letter "R" in front of the serial #. Left unit has a blue dot near volume, and "L" in front of serial #.

5 Adjusting Volume Control

Put gentle pressure against the volume control with your index finger. Slowly turn the control forward towards your nose for louder, and backwards toward back of head for softer.



DO NOT EXPOSE TO EXTREME HEAT
DO NOT GET WET

DO NOT SPRAY HAIR PRODUCTS WHILE SLEEP-EZE IS IN EAR



2 Proper Tip Selection

Two types of tips come with the unit:



Fan Style Tip



Mushroom Style Tip

Use whichever style is more comfortable. If the fit is not snug, remove the tip from the unit and put on another size, until your Sleep-eze feels snug and comfortable in your ear.

NOTE: The smallest Fan tip is already on the unit.

3 Installing The Battery

Open battery door with your fingernail. Remove one battery from the package and pull off tab. Insert the battery with the slightly beveled side fitting onto the battery door ledge, flat side up. The "+" mark on the battery door and the "+" mark on the battery should be visible. Use a size 10 battery.



6 Removing & Turning Off

Use removal cord only, to extract the Sleep-eze from your ear.



Open the battery door to turn the unit off.


7 Cleaning And Care

Use the brush or cleaning tool on a regular basis. Brush the tip of your Sleep-eze to clear away any wax. Use a cloth moistened with alcohol on the ear tip; dry the end.



4 Insertion Into Ear

(1st) Pull back & up to straighten ear canal

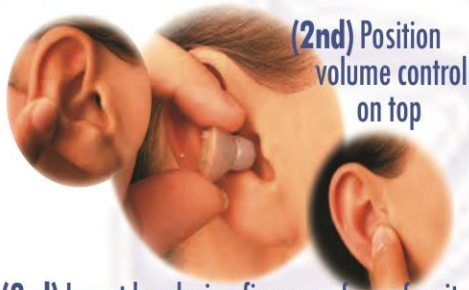

(2nd) Position volume control on top

(3rd) Insert by placing finger on face of unit, gently wiggling it into the ear.

8 Storage

Store Sleep-eze in the included storage pouch when not in use.



PRODUCT SPECIFICATIONS

Max SPL Output	81dB in a 2cc Cavity
Frequency	800hz to 6400hz
Drain	0.29mA
Battery Operating Hours	250 Max.*

* Based on use & climate conditions

WARNING: BATTERIES ARE DANGEROUS IF SWALLOWED

1. Keep batteries out of reach of small children and pets.
2. NEVER change batteries in front of children.
3. Never let children play with batteries.
4. Dispose of batteries properly. NEVER INCINERATE BATTERIES.
5. Never put batteries in your mouth for any reason.
6. ALWAYS check medication before swallowing, batteries have been mistaken for tablets.

If a battery is swallowed, immediately call emergency services at 202-625-3333 for counsel on treatment

Have a Good Night's Sleep.

Sleep-eze